



STILLE imagiQ3™ Legacy

Accessory catalogue

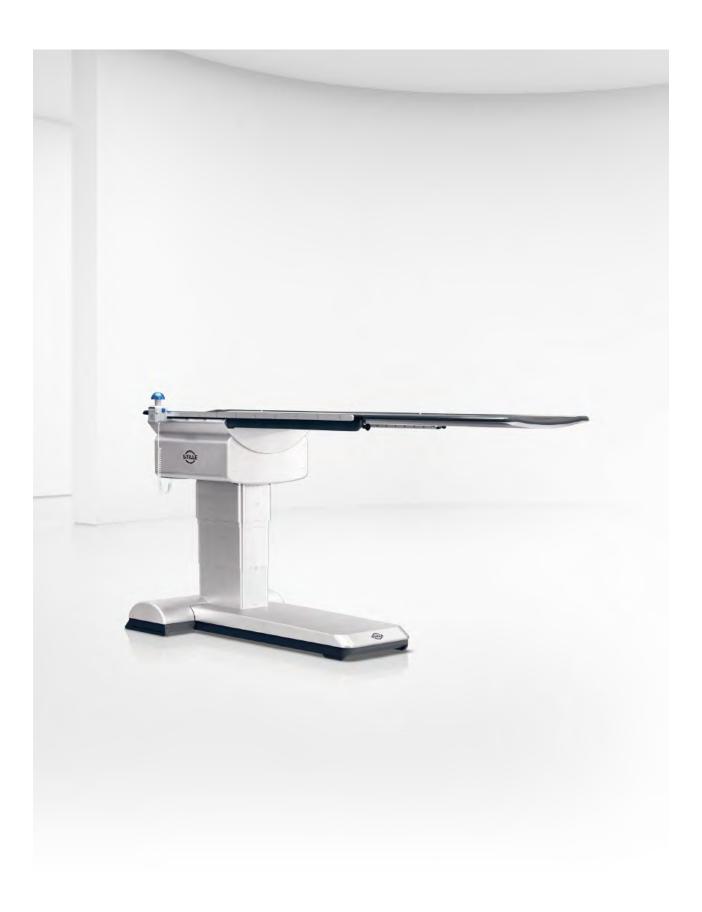


Table of contents

Introduction	04
Table configuration	06
Accessory packages	08
Detachable side rails	10
Anesthesia accessories	12
Infusion stand	12
Anesthesia frame	12
Arm, leg & foot supports	14
Armboards	14
Arm and leg guards	16
Leg holders	17
Foot support	18
Tibia support	18
Control device	20
Pan handle (additional pan handle)	20
General body supports & barriers	22
Lateral / side supports	22
Safety side rails	23
Patient restraints / leg straps	23
Headrests	24
Radiation protection	26
Surgical table extensions	28
Catheter trays	28
Instrument plate	29
Widening sections	29
Index	30

imagiQ3™ Legacy surgical table accessories offer a wide range of solutions for procedures such as vascular, electrophysiology, pain management, spine and ERCP.

Page | RANGE - CATEGORY OF THE ITEM

CATEGORY OF THE ITEM

Product name or specific characteristic(s)

Description of the item: use, characteristics, quick mounting instructions, material, etc.

Included:

Parts and numbers related to the item

Required:

Other accessories needed to mount or to use the item

Optional:

Alternative or suggestion of combination of items for a smoother workflow or optimal use

How to navigate through our catalogue?

All stored into one document for easy reference, imagiQ3™ Legacy accessories have been grouped and presented individually as illustrated below.



Illustration of the item. May not be an exact representation of the product.



Order information

EU: XX-XX-XXXX
US: XX-XX-XXXX

Part number* of the item for your order. Part number might be different depending on the region. Most regions outside the USA use the EU version. Products may not be available in all regions. Please contact your STILLE-representative for more information on market availability.

* To search by part number, simply refer to the index at the end of the catalogue.

TABLE CONFIGURATION









imagiQ3™ Legacy standard configuration

A mobile and highly radiolucent operating table with the patented True Free Float $^{\text{TM}}$. This Low-Dose Enabler $^{\text{TM}}$ is designed to optimize C-arm and O-arm use in helping to reduce scattered radiation and enhance image quality.



Order information

imagiQ3™ Legacy

- US, up to 10-day battery autonomy* (3-battery system): 550-1700
- EU, up to 5-day battery autonomy* (2-battery system): 550-1702
- EU, up to 10-day battery autonomy* (3-battery system): 550-1703
- * in normal use

Included:


- 1x "Slow Recovery" mattress
- 1x Pan handle for the floating tabletop
- 1x Wired hand control
- 1x Spare hand control
- 1x Emergency Trend Controller
- 2x Fixed US or EU/DIN side rails, 142.5 cm / 56.1" each
- 2x Ergonomic transport handles
- 4x Retractable and swiveling wheels
- 1x Power cord

Lithium-ion batteries:

3x for 550-1700 and 550-1703

2x for 550-1702

ACCESSORY PACKAGES

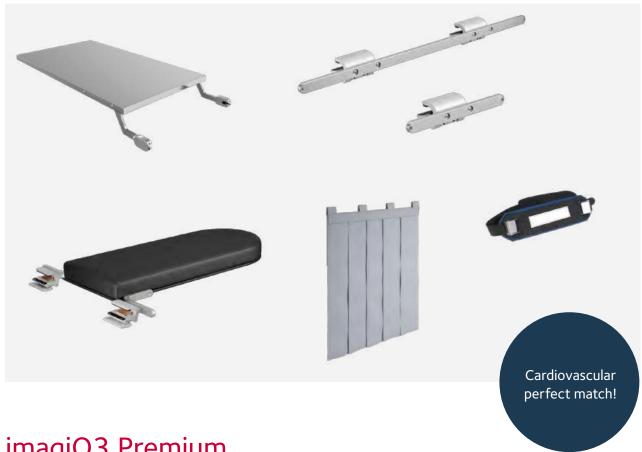

imagiQ3 Premium Vascular Package (US)

A complete cardiovascular procedure pack consisting of accessories for using with imagiQ3 Legacy. Exists without the Catheter Tray: imagiQ3 Basic Vascular Package (US). Only available in the USA as a package; items can be ordered individually.

Included:

One detachable long side rail
One detachable short side rail
One radiolucent fistula armboard with mattress
One radiation shield loop (grey version)
One patient strap
One 39.4" / 1m long catheter tray

Required:

Optional:

_



EU: Not available **US:** 555-1791 (Premium)

555-1790 (Basic, without catheter tray)

Included accessories

Detachable side rail (31.7" / 80.5 cm, single) page 10

EU: 535-1721 **US**: 535-1720

Fistula armboard

page 14 **EU**: 535-1730 **US**: 535-1730

Radiation shield loop grey

page 26 **EU**: 535-1772 **US**: 535-1772

Side rail short (11.2" / 28.5 cm, single)

page 11 **EU**: 535-1725 **US**: 535-1724

Deluxe maximum restraint


page 23

EU: alternative 514-100-465

US: O-RSX2

Catheter tray 1,000 mm

page 28 **EU**: 555-1750 **US**: 555-1750

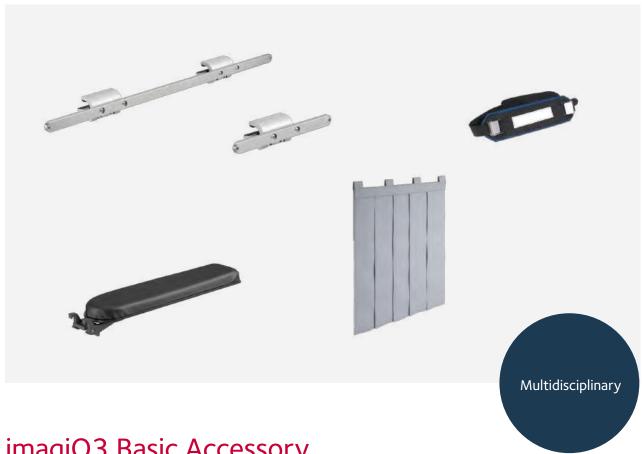

imagiQ3 Basic Accessory Packaging (US)

A multidisciplinary procedure pack consisting of basic accessories for using with imagiQ3 Legacy. Only available in the USA as a package; items can be ordered individually.

Included:

One detachable long side rail One detachable short side rail One basic armboard, with 2" pad One radiation shield loop (grey version) One patient strap

Required:

Optional:

-



EU: Not available US: 555-1793

Included accessories

Detachable side rail (31.7" / 80.5 cm, single)

page 10 **EU**: 535-1721 **US**: 535-1720

Basic armboard

page 15 **EU**: Not available **US**: 512-5000

Radiation shield loop grey

page 26 **EU**: 535-1772 **US**: 535-1772

Side rail short (11.2" / 28.5 cm, single)

page 11 **EU**: 535-1725 **US**: 535-1724

Deluxe maximum restraint

page 23

EU: alternative 514-100-465

US: O-RSX2

DETACHABLE SIDE RAILS

Side rail (70.0cm/27.5", single)


The most chosen detachable side rail for accessory mounting.

Not compatible with the 3D headrest.

Included:
One side rail

Required:

Optional:

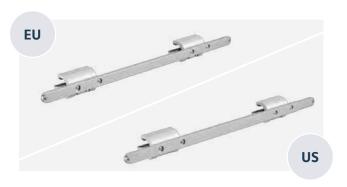

Order information

EU: 535-1729 **US:** 535-1723

Side rail (80.5cm/31.7", single)

Side rail enabling full table length side rail for imagiQ3 Legacy. Order two for mounting the fixed headrest

Not compatible with the 3D headrest and the adjustable headrest.

Included:
One side rail

Required:

Optional:

Order information


EU: 535-1721 **US:** 535-1720

Side rail short (28.5cm/11.2", single)

1-clamp detachable side rail to mount small accessories (max. 5kg: like the pan handle or infusion stand). Can be mounted on the free end of the imagiQ3 Legacy.

Not compatible with a retractor system.

- Included:
 One side rail
- Required:
- Optional:

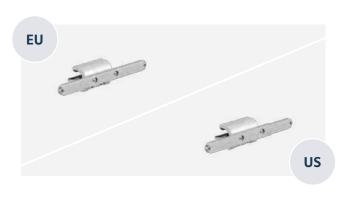



EU: 535-1725 **US:** 535-1724

ANESTHESIA ACCESSORIES

Infusion stand fixed

I.V. pole with three hooks with side rail fixation. Height is adjustable. Made in stainless steel and aluminum.



One I.V. pole with a non-detachable clamp

Required:

Fixed or detachable side rail

Optional:

-





Order information

EU: 514-100401 **US:** 514-100401-US

Anesthesia frame

Enables hanging drapes or equal material to differentiate the sterile zone from the non-sterile zone. Made in stainless steel and aluminum with polyurethane coating.

Included:

One flexible frame with two fixed clamps

Required:

Fixed or detachable side rails

Optional:

_





Order information

EU: 514-100311 **US:** 514-100311-US



imagiQ3 Legacy table with two 70 cm / 27.6" side rails, infusion stand, anesthesia frame, adjustable armboard, adjustable headrest and radiation shield loop.

^{*} Products may not be available in all regions. Please contact your STILLE-representative for more information on market availability.

ARM, LEG & FOOT SUPPORTS

Armboard, adjustable

The armboard is designed to support a patient's arm, ideal for IV. It is very easy to adjust in any desired position with one hand. This armboard is not radiolucent.



One armboard with a non-detachable clamp and an antistatic mattress

Required:

Fixed or 2-clamp detachable side rail

Optional:





Order information

EU: 514-50-3SR025 **US:** 514-50-3SR027

Fistula armboard

A non-adjustable radiolucent armboard for use in supporting the arm of a patient during imaging to provide an unobstructed image. The board is 81.1cm long x 35.0 cm wide (31.9" x 13.8").

Included:

One radiolucent board with mattress and two clamps

Required:

Optional:





Order information

EU: 535-1730 **US:** 535-1730

Basic armboard (US)

Radiolucent armboard with a simple design. Only available in the USA.

- Included:
 - One board, clamp and mattress
- Required:
- Fixed or 2-clamp detachable side rail
- Optional:





EU: Not available **US:** 512-5000



imagiQ3 Legacy table with 70 cm / 27.6" side rail, adjustable armboard and radiation shield loop*.

^{*} Products may not be available in all regions. Please contact your STILLE-representative for more information on market availability.

Arm and leg guard, curved

Arm and leg guard intended to support and keep the patient's arm and leg on the OR-table during surgery and transport. The curve bends outward 5.3 cm (i.e. orthogonal distance between the turning point of the curve and the mattress).



One piece of curved guard

Required:

Optional:

Cover for Arm & leg guard curved

EU/US: 514-100553





Order information

EU: 514-100552 **US**: 514-100552

Arm and leg guard, straight

Arm and leg guard intended to support and keep the patient's arm and leg on the OR-table during surgery and transport. The guard slides under the table pad.

Included:

One piece of guard

Required:

Optional:

Cover for Arm & leg guard straight

EU/US: 514-100555





Order information

EU: 514-100554 **US:** 514-100554

Leg holder with clamp (pair)

Leg supports intended for support and positioning of patient lower legs during surgery.

- Included: Two leg rests with clamps and mattresses
- Required:
 Fixed or 2-clamp detachable side rail
- Optional:





EU: 514-50-3SR032 **US:** 514-50-3SR033



imagiQ3 Legacy table with 3D headrest, curved arm and leg guard and radiation shield loop (grey version)*.

^{*} Products may not be available in all regions. Please contact your STILLE-representative for more information on market availability.

Foot support (pair)

The foot support is designed to provide stability for patients, especially during anti-Trendelenburg positioning. It can be adjusted in height and along the operating table to accommodate patients of different sizes.



Two supports, mattresses and clamps

Fixed or two 2-clamp detachable side rails

Optional:





Order information

EU: 514-100296 **US:** 514-100296

Tibia support (excluding clamp)

The tibia support is designed to support the knee or the foot of a patient during knee or leg surgery. Made in stainless steel and includes an antistatic cushion. Cushion diameter is 13.5 cm / 5.3", length 27 cm / 10.6".

Included:

Metal pole, mattress. The clamp is not included.

Fixed or 2-clamp detachable side rail + one separate clamp (see page 19)

Optional:





Order information

EU: 514-100367-K **US:** 514-100367-K

Radial clamp



A robust clamp to mount accessories like the tibia support.

Radial clamp MIX



A robust clamp to mount accessories like the tibia support. Has easy side rail attachment.



Order information

EU: 514-100305 **US:** 514-100305-US

Order information

EU: 514-100307 **US:** 514-100307-US



One stainless steel clamp

Required:

Optional: Tibia or lateral support (page

18 or page 22)

Included:

One stainless steel clamp

Required:

Optional: Tibia or lateral support (page 18 or page 22)

Accessory clamp



A simple clamp to mount accessories like the tibia support.



Order information

EU: 514-100301 **US:** 514-100301-US



clamp

Required: One aluminium

Optional:

Tibia or lateral support (page 18 or page 22)

CONTROL DEVICE



Pan handle (additional pan handle)

Allows for panning the table from different sides. Add a second pan handle to operate alongside the standard pan handle.

Included:
One wired pan handle


Required:
Fixed or detachable side rail

• Optional:



Order information

EU: 555-1760 **US:** 555-1761

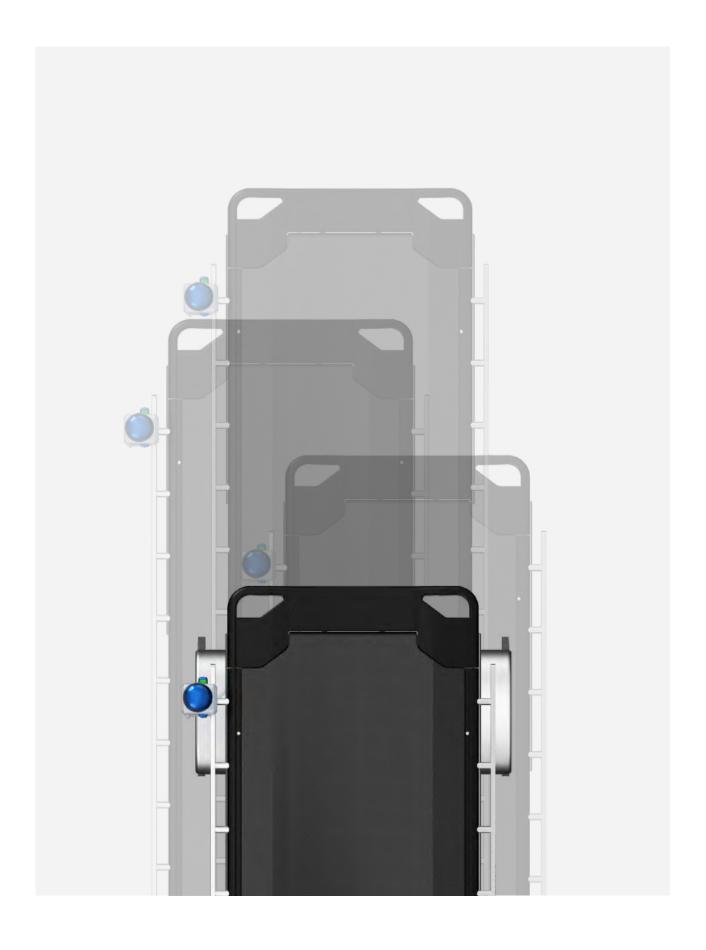

GENERAL BODY SUPPORTS & BARRIERS

Lateral support complete w clamp

The wide lateral support can be adjusted in height, longitudinal and lateral, depending on how the patient shall be positioned. Dimensions of the curved cushion 20x20 cm / 7.9" x 7.9".



One metal pole, clamp, curved mattress

Required:

Fixed or 2-clamp detachable side rail

Optional:

_





Order information

EU: 514-50-3SR028 **US:** 514-50-3SR030

Lateral support, adjustable excl. clamp

Lateral support with narrow pads, one plane and one curved, for positioning a patient on the table. Rotatable in different horizontal angles. The length of post is 35 cm / 13.8" and, the metal support of the pad is 17.5×7 cm / 6.9" x 2.8".

Included:

One rotatable support with two mattress models (one curved and one flat), no clamp

Required:

Fixed or 2-clamp detachable side rail + one separate clamp (see page 19)

Optional:

_





Order information

EU: 514-100560 **US:** 514-100560

Safety side rail, foldable/pair

Side gates designed for supporting a patient. They particularly ensure the patient's positioning and comfort during ERCP procedures. They can be easily folded down and when folded to upright position, lock automatically.



Two side gates, two non-detachable clamps

Required:

Fixed or 2-clamp detachable side rail(s)

Optional:

Cover, safety side rail single unit **EU only**: 514-100530-H





Order information

EU: 514-100530 **US:** 514-100530-US

Leg strap (EU)



2.1m long x 8.5 cm wide (82.7" x 3.3"), with Velcro® fastening, to secure the patient.



Order information

EU: 514-100465 **US:** Not available

Deluxe maximum restraint (US)



Soft padded, 24" long x 6" wide, with adjustable strap: 24"-80". To secure the patient.



Order information

EU: Not available **US:** O-RSX2

- Included: One patient restraint
- Required:
- Optional: Side rails or safety side rails (pages 10, 23)
- Included: One patient restraint
- Required:
- Optional: Side rails or safety side rails (pages 10, 23)

HEADRESTS

Headrest adjustable

Fully radiolucent headrest that extends the table length by 30 cm / 11.8". Ensures easy intubation and superior carotid visualization. Does not require side rails. Can be attached to the free end of the table only.

Not compatible with 80.5 cm / 31.7" long side rails.

Included:

One headrest with mattress and two fixed clamps

Required:

Optional:





Order information

EU: 535-1742 **US:** 535-1742

Headrest fixed

Radiolucent non-adjustable headrest that extends the table length by 30 cm / 11.8". Can be attached to the free end of the table only, with two detachable 80.5 cm / 31.7" side rails.

Included:

One head rest with mattress

Required:

Two 80.5 cm / 31.7" detachable side rails

Optional:





Order information

EU: 535-1741 **US:** 535-1741

3D Headrest (extension)

Carbon fiber headrest that extends the radiolucent length by 30 cm / 11.8". Useful for 3D rotation, angulation and oblique views.

Not compatible with fixed or detachable side rails with 2 clamps (\geq 70.0 cm / 27.5").




Required:

Optional:





EU: 535-1745 **US:** 535-1745

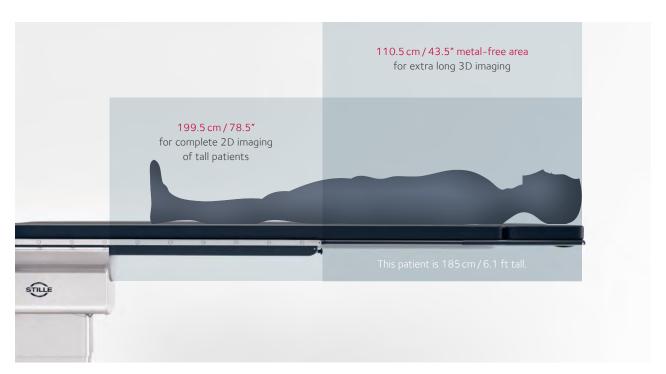

imagiQ3 Legacy table with 3D headrest / extension.

^{*} Products may not be available in all regions. Please contact your STILLE-representative for more information on market availability.

RADIATION PROTECTION

Radiation shield loop

X-ray protective with lead rubber (0.5 Pb) that protects against scattered radiation. Attach to side rails or directly to the tabletop with Velcro® strips.

Included:

5 protective panels that are fixed to each other

Required:

Optional:

Fixed or detachable side rail Radiation shield loop blue

EU/US: 535-1773





Order information

EU: 535-1772 (grey version) **US:** 535-1772 (grey version)

Radiation shield kit

X-ray protective with lead rubber (0.5 Pb) that protects against scattered radiation. Attach to the tabletop with strips of adhesive Velcro[©] that are placed on the tabletop. The kit can be used with or without side rails.

Included:

2 panels with cutout for the side rail clamps and 4 straight panels without cutout

Required:

Optional:

Radiation shield kit blue **EU/US**: 535-1771





Order information

EU: 535-1770 (grey version) **US:** 535-1770 (grey version)



imagiQ3 Legacy table with two $80.5 \, \text{cm} / 31.7$ " side rails, an adjustable armboard, one radiation shield kit (blue version) and two radiation shields loop (blue version).

^{*} Products may not be available in all regions. Please contact your STILLE-representative for more information on market availability.

SURGICAL TABLE EXTENSIONS

Catheter tray 1,000 mm (imagiQ3)

A stainless steel plate that provides an extended workspace. The plate dimensions are 1 m x 61.5 cm / 39.4" x 24.0".



One tray 1 m / 39.4"

Required:

Fixed or two 2-clamp detachable side rails

Optional:





Order information

EU: 555-1750 **US:** 555-1750

Catheter Tray 500 mm (imagiQ3)

A stainless steel plate that provides an extended workspace. The plate dimensions are 50 x 61.5 cm / 19.7" x 24.0".

Included:

One tray 50 cm / 19.7"

Required:

Fixed or two 2-clamp detachable side rails

Optional:





Order information

EU: 555-1751 **US:** 555-1751

Instrument plate with clamp

Adjustable tray to place instruments and other devices for the surgical team.

Included:

One instrument tray and clamp

Required:

Fixed or detachable side rail

Optional:

_ .





Order information

EU: 514-50-3SR036 **US:** 514-50-3SR037

Widening sections, 455mm pair EU

Two 45.5 cm / 17.9" long pieces. Widens the tabletop by 10.4 cm / 4.1" for ERCP or bariatrics.

Included:

Two sections, clamps and mattresses

Required:

Fixed or 2-clamp detachable side rail(s)

Optional:

_





Order information

EU: 514-100422 **US:** Not available

Index

Part number	Product name	Page
O-RSX2	Deluxe maximum restraint (US)	23
512-5000	Basic armboard (US)	15
514-100296	Foot support (pair)	18
514-100301	Accessory clamp, EU	19
514-100301-US	Accessory clamp, US	19
514-100305	Radial clamp (EU)	19
514-100305-US	Radial clamp, US	19
514-100307	Radial clamp MIX (EU)	19
514-100307-US	Radial clamp MIX, US	19
514-100311	Anesthesia frame (EU)	12
514-100311-US	Anesthesia frame, US	12
514-100367-K	Tibia support (excluding clamp)	18
514-100401	Infusion stand fixed (EU)	12
514-100401-US	Infusion stand fixed, US	12
514-100422	Widening sections, 455 mm pair EU	29
514-100465	Leg strap (EU)	23
514-100530	Safety side rail, foldable/ pair EU	23
514-100530-H	Cover, safety side rail single unit (EU)	23
514-100530-US	Safety side rail, foldable/ pair US	23
514-100552	Arm and leg guard, curved	16
514-100553	Cover, arm and leg guard curved	16
514-100554	Arm and leg guard, straight	16
514-100555	Cover, arm and leg guard straight	16
514-100560	Lateral support, adjustable excluding clamp	22
514-50-3SR025	Armboard, adjustable DIN (EU)	14
514-50-3SR027	Armboard, adjustable USA	14
514-50-3SR028	Lateral support complete w clamp EU	22
514-50-3SR030	Lateral support complete w clamp US	22

Part number	Product name	Page
514-50-3SR032	Leg holder with clamp EU (pair)	17
514-50-3SR033	Leg holder with clamp US (pair)	17
514-50-3SR036	Instrument plate with clamp (EU)	29
514-50-3SR037	Instrument plate with clamp US	29
535-1720	Side rail US (80.5 cm / 31.7", single)	10
535-1721	Side rail DIN (EU, 80.5 cm / 31.7", single)	10
535-1723	Side rail US (70.0 cm / 27.6", single)	10
535-1724	Side rail short US (28.5 cm / 11.2", single)	11
535-1725	Side rail short DIN (EU, 28.5 cm / 11.2", single)	11
535-1729	Side rail EU (70.0 cm / 27.6", single)	10
535-1730	Fistula armboard	14
535-1741	Headrest fixed	24
535-1742	Headrest adjustable	24
535-1745	3D Headrest (extension)	25
555-1750	Catheter tray 1,000 mm (imagiQ3)	28
555-1751	Catheter tray 500 mm (imagiQ3)	28
535-1770	Radiation shield kit grey	26
535-1771	Radiation shield kit blue	26
535-1772	Radiation shield loop grey	26
535-1773	Radiation shield loop blue	26
555-1760	Pan handle EU (additional pan handle)	20
555-1761	Pan handle US (additional pan handle)	20
555-1790	imagiQ3 Basic Vascular Package (US)	08
555-1791	imagiQ3 Premium Vascular Package (US)	08
555-1793	imagiQ3 Basic Accessory Package (US)	09

STILLE – Surgical perfection. For life.

STILLE is one of the oldest medical device companies in the world with over 180 years' experience. At STILLE, we develop, produce and sell surgical instruments and surgical tables to physicians worldwide.

Surgical Tables

We offer specialty tables for minimally invasive procedures, with safety and features optimized for both in- and outpatient settings.

The STILLE Medstone mobile surgical imaging tables contribute to enhance efficiency in the OR, all while representing value-priced solutions with both quality and reliability.

The STILLE imagiQ3™ surgical imaging table with its low-dose capabilities, is optimized for advanced mobile Hybrid OR settings as well as peripheral and interventional procedures.

Surgical Instruments

We have a complete offering of handcrafted, forged instrumentation for all fields of open surgery, with a special emphasis on cardiovascular surgery, reconstructive and aesthetic plastic surgery.

Our instruments feature delicate design, industry leading durability and a unique feel, which has made STILLE a well-known and highly respected supplier by surgeons throughout the world.

Learn more at www.stille.se

STILLE AB

Ekbacken 11

SE-644 30 Torshälla Sweden

+46 8 588 580 00

www.stille.se

STILLE Surgical Inc.

840 Parkview Blvd Lombard, IL 60148 United States

1.800.665.1614

www.stille.se/us



Surgical perfection. For life.



These products may not be available in all regions. Please contact your STILLE-representative for more information on market availability.

This document correctly describes the devices at the time the document was printed. Please always refer to the user manual for any product amendments.

The product images shown are for illustration purposes only and may not be an exact representation of the product.